




Infinity-Lock™ 3 mm and 5 mm

COMPLEX KNEE INJURIES – SIMPLE SOLUTIONS.

The Infinity-Lock™ 3 mm and Infinity-Lock 5 mm are non-absorbable, sterile, poly(ethylene terephthalate) flat suture tapes.

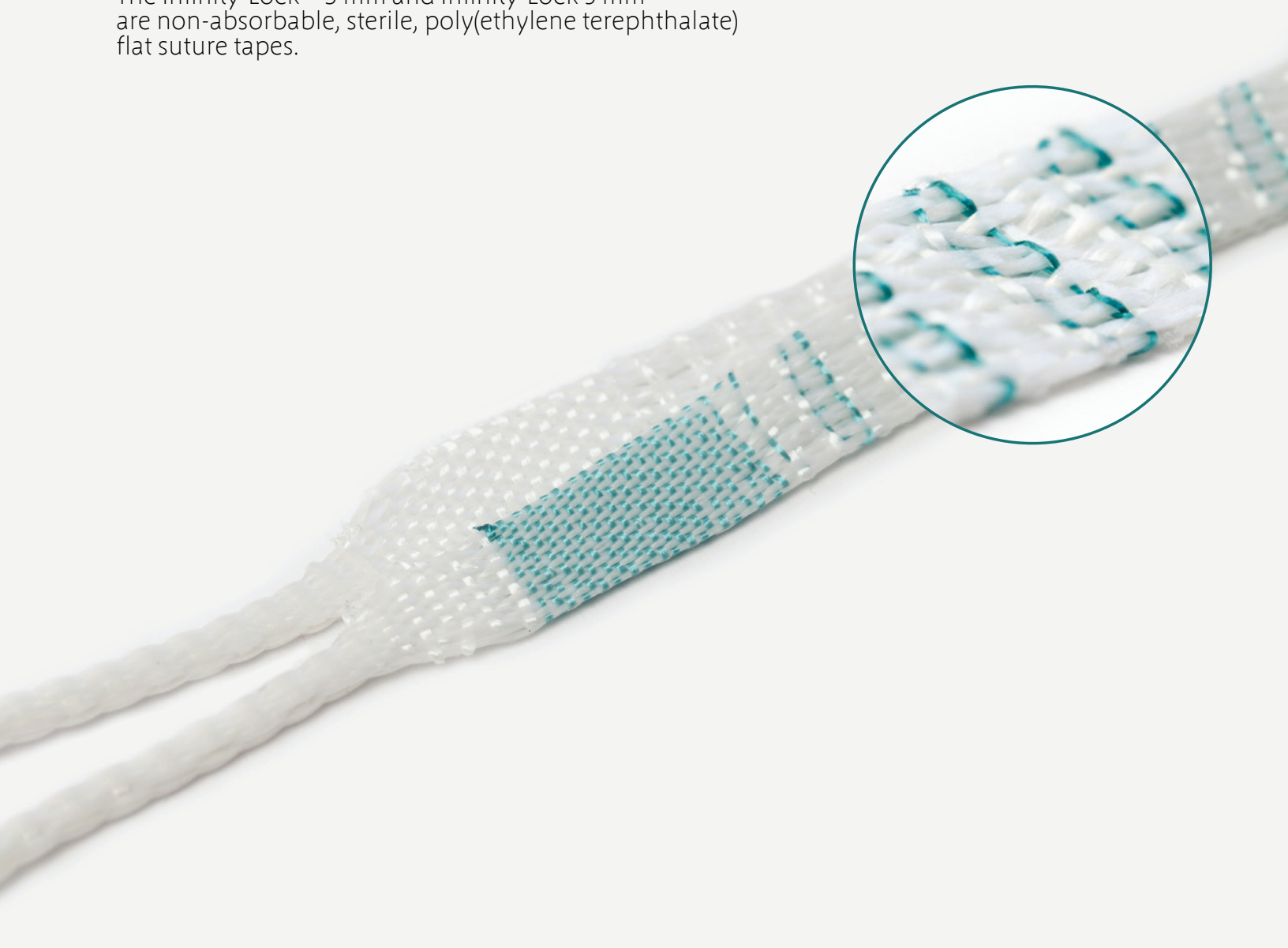

CONFIDENCE TO PERFORM

STRONG:

3 MM TAPE STRENGTH 800 N / 5 MM TAPE STRENGTH 1000 N

LOW STRETCH UNDER CYCLIC TENSION

The Infinity-Lock™ 3 and Infinity-Lock 5 are prepared from fibers of high molecular weight, long-chain, linear polyesters having recurrent aromatic rings as an integral component. Infinity-lock tapes differ from USP suture by being in the form of woven tapes and in exceeding all USP sizes.

Infinity-Lock tapes come in two sizes, 3 mm and 5 mm. The tapes have an integral loop at one end which is used to facilitate a hitch through one end of a ruptured tendon. The tapes have two tails. One tail end has an identifying mark and both taper into a cord to assist with passing through soft tissue.

INDICATIONS

The Infinity-Lock 3 and 5 mm tapes are intended to be used for soft tissue approximation, including:

Soft tissue approximation

Achilles tendon repair Repair of Acute ruptures of the Achilles Tendon

HOW DOES IT WORK?

The Infinity-Lock 3 mm and Infinity-Lock 5 mm tapes are used to approximate disrupted soft tissues. The ends of the Infinity-Lock tapes are tapered to cords facilitating the passage of the flattened tape through tissue. The Infinity-Lock tape ends are passed through the adjoining tissue, with tension applied to bring the two tissue surfaces together. The surgeon can use his preferred construct to achieve reduction. The tape ends are secured with the surgeon's preferred technique.

ORDER CODES

102-1100 Infinity-Lock 3 mm Tape (sterile, single-use)

102-1101 Infinity-Lock 5 mm Tape (sterile, single-use)

Contact **Xiros™** Inc. on enquiries@xirosna.com for further details or visit our website at www.xirosna.com

We are registered with the FDA in the USA,
Registration number 3015525389, FEI Number 3015525389.

Developed and manufactured by **Xiros™** Ltd.
Springfield House, Whitehouse Lane, Leeds LS19 7UE

Xiros, and Infinity-Lock are trademarks of Xiros. All rights reserved © Xiros™ 2020.
Worldwide patents and patents pending.

The information above MUST NOT be used to perform the surgical procedure. Always refer to the Instructions for Use LAB 272 which contains detailed information and essential warnings and precautions.

DOC-E7-003 Rev A



Get involved

At Xiros we are constantly striving to improve outcomes for patients, and we want to hear from you.

Help us create the future of sports medicine and share your opinions and expertise with us at:

www.xirosna.com